



360 AIR WALKER

INSTRUCTIONS FOR USE

Single Patient Use Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

- Grade 2 & 3 Ankle Sprains
- Stable Fractures
- Post-Operative Use
- Trauma
- Rehabilitation

WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CLEANING INSTRUCTIONS:

1. Remove the liner from the walker.
2. Hand wash liner using a mild detergent and cold water.
3. Lay liner flat and air dry ONLY.
4. Plastic walker shell can be washed with a damp cloth and mild detergent.

WARRANTY:

Manufacturer guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.

PART NUMBER:

Part Number	Size	Men's Shoe Size	Women's Shoe Size
16103	SM	5 – 6	5.5 – 7
16105	MD	6.5 – 10	7.5 – 11
16107	LG	10.5 – 12	11.5 - 13

EUROPEAN REPRESENTATIVE:




APPLICATION INSTRUCTIONS

1. While seated, release straps, remove front shell, and place leg into the walker. Wrap liner around the foot and fasten securely.
2. Wrap liner around leg and fasten securely. Ensure that the air bulb is centered over the leg.
3. Reattach front shell and fasten securely.
4. Pull strap over front shell and slide strap through the D-ring.
5. Pull on the strap to add tension and fasten securely. Fasten all three straps following steps 4 & 5.



Inflating/Deflating the Air Bladder

6. The knob directly below the air bulb controls the air bladder. Turn the knob clockwise to close the valve, and turn the knob counter-clockwise to open the valve.
7. To inflate the air bladder, turn the knob clockwise and pump the air bulb until a snug fit is achieved. Be sure not to over-inflate. Deflate after each use by turning the knob counter-clockwise.

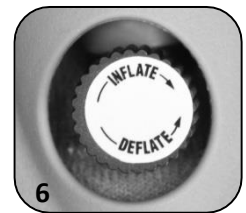

United Ortho Enterprises

2235 Pennsylvania St. Fort Wayne, IN 46803
 Phone: 800-227-8748; Fax: 877-688-7975
 www.United-Ortho.com