



PREMIUM KNIT SUPPORT, ANKLE INSTRUCTIONS FOR USE

Single Patient Use Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

- Posttraumatic irritation
- Ligament weakness
- Tendonitis/Arthritis
- Swelling from arthritis and osteoarthritis

WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

- Never wear product over an open wound.
- Do not sleep in this product.
- Do not use cream or lotions under this product.

CLEANING INSTRUCTIONS:

1. Clean material in warm water using mild soap.
 - a. Do NOT use soap with lotion.
 - b. Rinse thoroughly.
 - c. If not rinsed thoroughly, residual soap may cause irritation and affect the life of the product.
2. Lay flat and air dry ONLY.

WARRANTY:

Manufacturer guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.

PART NUMBER: 900250-XX.

-XX	Size	Circumference 1" above malleolus
-01	XS	6.75" - 7.50"
-03	SM	7.50" - 8.25"
-05	MD	8.25" - 9.00"
-07	LG	9.00" - 9.75"
-08	XL	9.75" - 10.50"
-09	2XL	10.50" - 11.25"

APPLICATION INSTRUCTIONS:

1. Use two hands to firmly grasp sides of support as shown. Pull support over foot.



2. Adjust support so that each visco pad sits below and behind malleolus on each side of the ankle.



To remove support:

- Roll top of support down.
- Pull at bottom of brace and remove.

EUROPEAN REPRESENTATIVE:



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Phone: +31.70.345.8570
Fax: +31.70.346.7299

UNITED SURGICAL
2235 Pennsylvania St.
Fort Wayne, IN 46835
USA

United Ortho Enterprises

2235 Pennsylvania St. Fort Wayne, IN 46803
Phone: 800-227-8748; Fax: 877-688-7975
www.United-Ortho.com